

STEAM•DOT™ PROCESS INDICATORS

FOR STEAM STERILIZATION

Reorder No.: **36400100**

Instructions for use

Propper Manufacturing Company Steam-Dot Process Indicator for steam sterilization is designed for use by a health care provider to demonstrate that the unit or load has been exposed to a steam sterilization process, and to distinguish between processed and unprocessed units or loads. The indicator dots turn from white to dark brown/black when exposed to steam sterilization conditions, thus providing an indication of processed items.

Each indicator consists of the reactive steam indicator ink printed on a substrate paper circle, 3/4" in diameter, with adhesive backing. One thousand (1000) individual indicators on a roll of paper placed in a paper dispenser box with the following dimensions: 4 3/4" x 4 3/4" x 1". The unprocessed indicator color is white.

PERFORMANCE


Steam-Dot Process Indicator conforms to the requirements of ANSI/AAMI/ISO 11140-1:2014 for Type 1 process indicators.

HOW TO USE STEAM-DOT™ PROCESS INDICATOR

The indicator can be used in

- Gravity cycles: 121°C/250°F
- Pre-vacuum cycles: 132°C/270°F
- Pre-vacuum cycles: 134°C/273°F
- Pre-vacuum cycles: 135°C/275°F

1. Remove an indicator from the dispenser box.
2. Affix the Steam-Dot Process Indicator to the outside of the sterilization container, cassette, pouch or other package prepared for sterilization.
3. Process the sterilization load.
4. After processing examine Steam-Dot indicators. The color change to black or dark brown is an indication that the item passed through a steam sterilization cycle, but it does not guarantee that adequate sterilization conditions have been met. The indicator maintains stable color after sterilization processing.

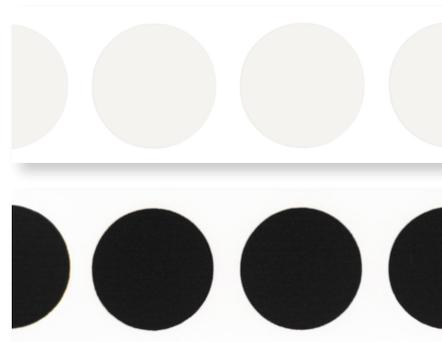

STORAGE CONDITIONS AND SHELF LIFE

Steam-Dot Process Indicator should be stored at ambient temperature at or below 90°F/32°C and at relative humidity below 70%.

The shelf life of Steam-Dot Process Indicator is 48 months from the date of production.



Keep Dry



Do not reuse



Consult instruction for use



36-04 Skillman Avenue
Long Island City, NY 11101
Tel. (718) 392-6650 • Fax. (718) 482-8909
www.propermfg.com