

## Before you mail this business reply envelope...

- Ensure all fields are completed on this envelope
- Enclose two (2) **spore test strips** (*processed in sterilizer*)

**To obtain your test results:** create an account at  
[www.proppermfg.com/lab](http://www.proppermfg.com/lab)

## Sterilization Report Form

Facility ID is used to locate your account. Please write clearly

1. Facility ID:

2. Date of test: <sub>M</sub> <sub>M</sub> / <sub>D</sub> <sub>D</sub> / <sub>Y</sub> <sub>Y</sub>

3. Temp:    °C / °F

4. Exposure time:    minutes

5. Steam  Ethylene Oxide  Dry Heat

6. Phone

Please use same phone number on each envelope

7. Contact/Operator name:

8. Sterilizer name:

9. Facility name:

Facility address:

Zip code:

## Quick Guide

*Full instructions for use on package*

1. Remove a **test pack** from the box and place in most difficult location to sterilize.
2. Sterilize the load per normal procedure.
3. Complete all the information on this envelope.
4. After processing, open the test pack, remove the gauze and extract the test strips. Do not mail the gauze or pouch.
5. Place the **spore test strips** into this envelope, seal, and mail.
6. **To obtain your results**, log in to your account at [www.proppermfg.com/lab](http://www.proppermfg.com/lab)